
HEINE Skia/Retinoscope Trainer

HEINE Optotechnik GmbH & Co. KG
Dornierstr. 6 · 82205 Gilching · Germany
E-Mail: info@heine.com · www.heine.com
med 13301 2020-06-02

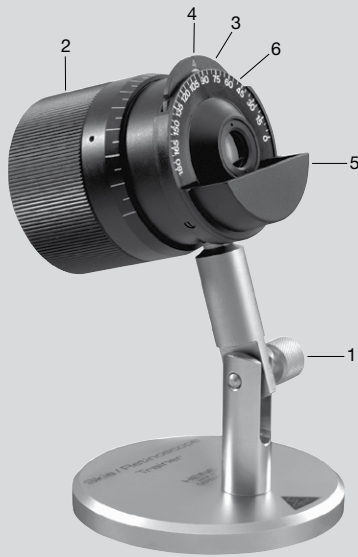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HEINE Skia/Retinoscope Trainer

Allgemeine Informationen

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General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

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HEINE Retinoscope Trainer

General Information

The HEINE Retinoscope Trainer is not intended for use on patients and is not a medical product according to directive 93/42 EWG/CEE/EEC.

The instrument is intended as a training device for the technique of retinoscopy. It can also be used as a reference device to check the function of instruments used for refraction measurement, provided the relevant optical principles are applied. It is not designed for ophthalmoscopy, instead of this, we recommend the Ophthalmoscope Trainer C-000.33.010.

We are unable to give an introduction to Retinoscopy and its application in refraction in these instructions. We recommend the published literature on this subject.

Technical Description

The optics of the trainer include an achromatic lens with a pupil aperture and an orange-coloured curved shell which can be moved along the optical axis.

Technical data

Weight:	600 g
Achromat:	f = 32 mm
Pupil size:	2, 3, 4, 5, 6, 8 mm

Using the instrument

Setting up the trainer: Adjust the body of the trainer so that the optical axis corresponds to your viewing axis, (angle can be adjusted by loosening screw 1).

Selecting refraction: Rotate the housing to select a value as shown on the scale (2). Select the desired value carefully, particularly around the 0 dpt position. View the scale from directly above to reduce the chance of error.

Selecting pupil size: Six different pupil sizes can be selected by rotating the selector wheel (3). Ensure that the desired value clicks into place. The selected pupil size is indicated (4).

Using trial lenses: A slot is provided in front of the lens to accommodate two trial lenses. The scale (6) gives an indication of the cyl. angle.

Retinoscopy: Please ensure that you view along the optical axis of the trainer to avoid getting a false result. You can check this by making sure that the two reflexes generated by the retinoscope on the front and back side of the trainer lens co-incide.

In general, the higher the refraction, the less precise is the result.

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