**HEINE** QUALITY  
MADE IN GERMANY

## ∴ HEINE Wall Charger CW1

 **HEINE Optotechnik GmbH & Co. KG**  
Dornierstr. 6 · 82205 Gilching · Germany  
E-Mail: [info@heine.com](mailto:info@heine.com) · [www.heine.com](http://www.heine.com)  
MED 234757      2021-04-16

V-200.00.329

# HEINE Wall Charger CW1

<b>DEUTSCH</b>	<b>3</b>
<b>ENGLISH</b>	<b>6</b>
<b>FRANÇAIS</b>	<b>9</b>
<b>ESPAÑOL</b>	<b>12</b>
<b>ITALIANO</b>	<b>15</b>
<b>SVENSKA</b>	<b>18</b>
<b>NEDERLANDS</b>	<b>21</b>
<b>DANSK</b>	<b>24</b>
<b>NORSK</b>	<b>27</b>
<b>SUOMI</b>	<b>30</b>
<b>PORTUGUÊS</b>	<b>33</b>

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 Please read and follow these instructions for use and keep them for future reference.

### Intended Use

The HEINE Wall Charger CW1 is intended only for charging the HEINE OMEGA 600 binocular indirect ophthalmoscope.

The product may only be used by qualified medical personnel and in professional healthcare facilities.

 **For U.S. only:**  
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

### Warnings and Safety Information

 **CAUTION!** This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

### Product overview

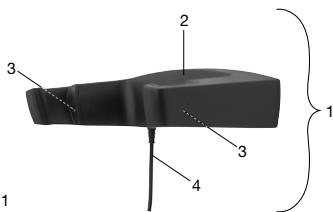
Fig. 1

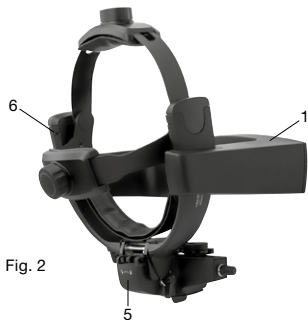
Fig. 2

- |                     |                           |
|---------------------|---------------------------|
| 1 Wall Charger CW1  | 4 Power supply            |
| 2 Contact indicator | 5 OMEGA 600               |
| 3 Charging pins     | 6 Charge status indicator |

### Assembly and disassembly

 To ensure all-pole disconnection from the mains at all times, the ME device must be installed so that the switch mode power supply is accessible and disconnectable.

Before mounting the device, check whether you need to use a special wall plug for the subsurface of your wall and whether the subsurface exhibits sufficient load-bearing capacity for the mechanical stress.

 The wall plugs supplied with the device are universal wall plugs and suitable for most building materials (e.g. concrete, solid brick, brick, plasterboard or gypsum cardboard).

Hold the drill template enclosed with your device horizontally in place on the desired position and mark the drill holes. Mount the Wall Charger CW1 (1) according to the enclosed package insert.

To disassemble, disconnect the Wall Charger CW1 (1) from the mains first, loosen the lower screw, push upwards and take off the wall.

## Setting up

### Primary plug

The primary country adapter of the power supply (4) is interchangeable, this enables worldwide operation. Fig. 3 and Fig. 4 describe the procedure of changing the plug.

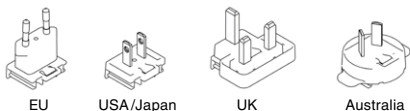
Fig. 3

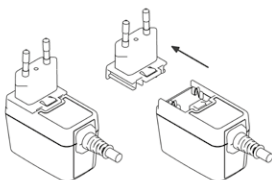
Fig. 4

⚠ After you have checked the mains voltage indicated on the nameplate of the device, connect the mains supply plug to the mains supply.

Mount the Wall Charger CW1 (1) in such a fashion, that the power supply (4) can be easily reached and unplugged. The LED indicates the operational readiness of the power supply (4). The contact indicator (2) briefly lights up blue when the mains supply plug is connected to the mains supply and the Wall Charger CW1 (1) is ready for operation.

To turn off the Wall Charger CW1 (1) just disconnect the power supply (4) from the mains supply. The Wall Charger CW1 (1) is intended for setting up and use in medical areas outside the patient environment (at least 1.5 meters from the patient or patient support pursuant to IEC 60601-1, see Fig. 5).

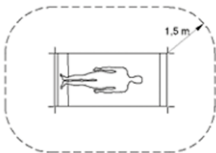
Fig. 5

## Operation

To charge, insert the binocular indirect ophthalmoscope OMEGA 600 (5) into the Wall Charger CW1 (1) as shown in Fig. 2. The contact indicator (2) on the Wall Charger CW1 (1) indicates contact with the OMEGA 600 (5). The charge status indicator (6) is located on the battery compartment of the OMEGA 600.

If the contact indicator (2) on the Wall Charger CW1 (1) or the charge status indicator (6) on the battery compartment of the OMEGA 600 (5) do not light up after the OMEGA 600 (5) is inserted into the Wall Charger CW1 (1), check that the OMEGA 600 (5) is inserted correctly and that the mains supply plug is connected to the mains supply.

## Hygienic reprocessing

The instruction is available at:

- [www.heine.com](http://www.heine.com)
- in a paper version which you can request from the address listed

## Maintenance

The device is maintenance free.

## Service

The device has no components serviceable by the end-user.

## General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

## General Warnings

 Do not use power supplies where damage has been detected.

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in presence of flammable gases / liquids, or in an oxygen rich environment.

This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Do not use the device outdoors.

Heating during operation is normal and harmless.

All load-bearing elements of the device are to be loaded only in the manner intended.

Dropping the unit or mounting accessories can cause injury and damage to property.

Check at regular intervals whether the wall mounting and the mounting accessories are in proper condition. In particular, make sure that all screws are firmly tightened.

Do not let the power cable become taut as this could damage the device or pose a tripping hazard.

## Disposal

 The product must be recycled as separated electrical and electronic devices.

 Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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## Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.	
Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
	The supply voltage quality should be that of a typical hospital environment.
	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Performance features of the ME system that have been determined to be essential to the performance	None
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used. The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.
Test	Compliance
RF emissions CISPR11	Group 1, Class B
Conducted emissions (EN 55011/CISPR 11)	Not applicable, because of internally powered mode
Radiated emissions (EN 55011/CISPR 11)	Passed
Harmonic current emissions (IEC 61000-3-2)	
Voltage changes, voltage fluctuations and flicker (IEC 61000-3-3)	
Immunity	See attached immunity test levels

Immunity test levels		
Test	Test level	
	IEC 60601-1-2 test levels	Compliance test levels
Electrostatic Discharge (IEC 61000-4-2)	Contact Discharge: $\pm 8$ kV Air Discharge: $\pm 2$ kV, $\pm 4$ kV, $\pm 8$ kV, $\pm 15$ kV	
Radiated RF EM fields (IEC 61000-4-3)	3 V/m 80–2700 MHz 80 % AM at 1 kHz	
Electrical fast transients / bursts (IEC 61000-4-4)	$\pm 2$ kV 100 kHz repetition frequency	
Surges (IEC 61000-4-5)	$\pm 0.5$ kV, $\pm 1$ kV	
Conducted disturbances induced by RF fields (IEC 61000-4-6)	3 V 0.15 MHz – 80 MHz 6 V in ISM bands between 0.15 MHz and 80 MHz 80 % AM at 1 kHz	
Proximity fields from RF wireless communications equipment (IEC 61000-4-3)	385 MHz; Pulse Modulation: 18 Hz; 27 V/m 450 MHz, FM: $\pm 5$ Hz deviation: 1 kHz sine; 28 V/m 710, 745, 780 MHz; Pulse Modulation: 217 Hz; 9 V/m 810, 870, 930 MHz; Pulse Modulation: 18 Hz; 28 V/m 1720, 1845, 1970 MHz; Pulse Modulation: 217 Hz; 28 V/m 2450 MHz; Pulse Modulation: 217 Hz; 28 V/m; 5240, 5500, 5785 MHz; Pulse Modulation: 217 Hz; 9 V/m	
Power frequency magnetic fields (IEC 61000-4-8)	30 A/m; 50Hz or 60 Hz	30 A/m; 60 Hz
Voltage dips (IEC 61000-4-11)	0 % U <sub>T</sub> ; 0.5 cycle; at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % U <sub>T</sub> ; 1 cycle and 70 % U <sub>T</sub> ; 25/30 cycles Single phase: at 0°	
Short interruptions (IEC 61000-4-11)	0 % U <sub>T</sub> ; 250/300 cycles	

**Technical specification**

Input	100–240V~/ 50–60Hz / 300–150mA
Power consumption	Max. 6 W
Protection class	Class II
Charging time	Typ. 1.5 h

**Environmental conditions**

Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa

**Erläuterung der verwendeten Symbole**  
**Explanation of utilized symbols**  
**Explication des symboles utilisés**  
**Explicación de los símbolos utilizados**  
**Spiegazione dei simboli utilizzati**  
**Förklaring av symboler som används**  
**Verklaring van de gebruikte symbolen**  
**Forklaring af de anvendte symboler**  
**Symbolforklaring**  
**Käytettyjen symbolien selitys**  
**Explicação dos símbolos utilizados**

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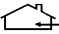
	<p>Nur in geschlossenen Räumen benutzen  For indoor use only  Utiliser uniquement dans des locaux fermés  Sólo utilizar en espacios cerrados  Utilizzzare solo in ambienti chiusi  Får endast användas i slutna rum.  Uitsluitend in afgesloten ruimten toepassen  Må udelukkende benyttes i lukkede rum  Skal kun brukes i lukkede rom  Käyttö on sallittu ainoastaan suljetuissa tiloissa  Utilizar apenas em espaços fechados</p>
	<p>Gebrauchsanweisung  Instructions for use!  Mode d'emploi  Manual de instrucciones  Istruzioni per l'uso  Bruksanvisning  Gebruiksaanwijzing  Brugsanvisning  Bruksanvisning  Käyttöohjeet  Instruções de utilização</p>
	<p>Gerät der Schutzklasse II  Class II equipment  Appareil de classe de protection II  Aparato de clase de protección II  Apparecchio di classe di protezione II  Apparat i skyddsklass II  Apparaat van beschermingsklasse II  Apparater i sikkerhedsklasse II  Klasse II-utstyr  Suojausluokan II laite  Aparelho da classe de proteção II</p>
	<p>The Regulatory Compliance Mark (RCM)  The Regulatory Compliance Mark (RCM)  Marque réglementaire de conformité (RCM)  The Regulatory Compliance Mark (RCM)  The Regulatory Compliance Mark (RCM)  The Regulatory Compliance Mark (RCM)  The Regulatory Compliance Mark (RCM)  The Regulatory Compliance Mark (RCM)  The Regulatory Compliance Mark (RCM)  The Regulatory Compliance Mark (RCM)  The Regulatory Compliance Mark (RCM)  Vaatimustenmukaisuusmerkki (RCM)  (RCM)- Marca de Conformidade Regulamentar</p>

<p>~</p>	<p>Wechselstrom (AC)  Alternating current (AC)  Courant alternatif (CA)  Corriente alterna (CA)  Corrente alternata (AC)  Växelström (AC)  Wisselstroom (AC)  Vekselstrøm (AC)  Vekselstrøm (AC)  Vaihtovirta (AC)  Corrente alternada (CA)</p>
<p>≡</p>	<p>Gleichstrom  Direct current (DC)  Tension continue  Tensión continua (CC)  Tensione continua  Likspänning  Gelijkspanning  Jævnspænding DC  Likestrøm dc  Tasajännite  Tensão contínua (CC)</p>