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 Please read and follow these instructions for use and keep them for future reference.

## Warnings and safety information

 **CAUTION!** This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations with moderate or severe extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice. Notes are important, but not related to hazardous situations.

## Intended use

The Protective case for visionPRO DS1 is exclusively intended for the use with the HEINE visionPRO video laryngoscope. The product may only be used by qualified medical professionals and in professional healthcare facilities.

## Operation

### Attaching and removing the protective case

To attach the optional protective case, attach the protective case to the top front of the DS1 display. Ensure that the opening for the display hinge is correctly orientated.

Carefully pull the display protection case downwards and pull it over the display frame of the visionPRO DS1.

To remove the protective case, proceed in reverse order.

### Charging of the visionPRO with attached protective case

 If the HEINE visionPRO is used with the protective case for the visionPRO DS1 display, the display has to be orientated upwards for charging in the visionPRO CS1 charging station. Check that the device is inserted correctly by determining whether the illuminated ring on the charging station is illuminated.

Fig. 1:  
visionPRO CS1 with correctly inserted  
visionPRO video laryngoscope

Fig. 2:  
visionPRO CS1 with incorrectly inserted  
visionPRO video laryngoscope

The operation of the video laryngoscope as well as of the charging station is described in a separate instruction of use.

## Hygienic reprocessing

The instruction is available:

- online at [www.heine.com](http://www.heine.com)
- in a paper version which you can request from the address listed

 If the HEINE visionPRO is used with the protective case for the visionPRO DS1 display, this must be removed before hygienic reprocessing. The protective case must also be hygienically reprocessed after each use.

## Maintenance

There is no regular maintenance required.

## Service

The device has no components serviceable by the end-user.

## General Notes

-  The protective case is a consumable article with a limited expected life cycle. Check regularly to ensure that the product is still in a safe and proper condition. If not, replace the protective case immediately.

## General Warnings

-  Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.  
Do not modify the device.

## Disposal

Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Technical specification
- Explanation of the used symbols

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**HEINE Optotechnik GmbH & Co. KG**